

10. Instruction For Use

CE 0123

Product Instruction for Metallic Bone Plate, sterile

**Distributed by: 7s Medical AG, Länggasse 4, 6208 Oberkirch, Switzerland, www.7smedical.com,
info@7smedical.com, Tel: 0041 41 925 66 60**



Manufacturer Name: Changzhou Waston Medical Appliance Co., Ltd.

Add: No.9Xihu Road, Wujin Hi-Tech Industry Zone, Changzhou, Jiangsu, PEOPLE'S REPUBLIC OF CHINA Website:
www.wastonchina.com E-mail: wastonchina@yahoo.com info@wastonmed.com
Tel: (+86)519-86522226 Fax: (+86)519-85135555



Name: Shanghai International Holding Corp. GmbH (Europe)

Add: Eiffestrasse 80, 20537 Hamburg, Germany
Tel. +49-40-2513175 Fax. +49-40-255726
E-mail: shholding@hotmail.com

1 . Product Name

Metallic bone plate

2. Function, Structure and Components

Metal Bone Plates are designed according to bionics of the structural characteristics of human skeleton by using a three coordinate measuring instrument. Its shape, sizes, hole number, hole diameter and spacing, are determined by the different fracture parts used in the body and its weight-bearing situations. The screw type of plate is ball and cone.

The implantation method of normal metallic bone plates is that partial incision, direct placement, using compression technique. In order to make the plate fit with bone surface, the plate needs to be pre-bended, there must be a good anchor synergy between the screw and bone. Screws must be bicortical fixation stability depending on the friction produced by close attachment between plates and bone surface; generally the implantation of locking plates uses small incision minimally invasive and subcutaneous insertion, using bridging technology, no need to be pre-bended precisely and stucked closely to bone surface, its stability is not completely dependent on the anchor, screws can be unicortically fixed, the stability are from the locking force between plates and screws.

Products are made of pure titanium (TA3), which in strict accordance with the requirements of ISO5832-2: 1999(E), "Implants for surgery -- Metallic materials -- Part 2: Unalloyed titanium" and Titanium Alloy (TC4), which in strict accordance with the requirements of ISO5832-3: 1996(E) "Implants for surgery -- Metallic materials -- Part 3: Wrought titanium 6-aluminium 4-vanadium alloy".

The final product delivery status is sterile packaging.

3. Type & Specifications

Please see Annex I for detailed product list and specifications.

4. Intended for Use

The metallic bone plate is intended to be used for bone fracture internal fixation.

5. Indication

Metallic bone plate are is used together with bone screws, to for connection and fixation of the bone fracture fragments such as part, for limbs, clavicle bones, rib and or sternum.

6. Contraindication

- a) Aged osteoporosis patients, the fractures not have sufficient integrity.
- b) Soft tissue infections and defects, disorders of calcified tissue metabolism and immune system, systemic neurological disease.
- c) Those who are allergic to the metal and do not comply with the doctor's orders.

7. Complications

- a) Allergic or rejection reactions to metal;
- b) Limb contraction due to fracture compression or bone resorption;
- c) Discomfort, pain or paresthesia generated by implants;
- d) Loosening or bending of the implants may cause nonunion or delayed healing of the fracture and bony defect;
- e) Any of the above symptoms after surgery is required for early revision surgery;
- f) Osteonecrosis;
- g) Nonunion or delayed healing will cause implant fracture.

8. Notes

- a) The delivery status of the Intramedullary Nail is sterile. It is sterilized with γ ray.
- b) Check packaging's integrity before surgery. If there is any damage, it cannot be used.
- c) Doctors should perform the preoperative assessment of patients, including the weight, occupation, activity intensity, mental status and allergy history, then determine whether it is suitable for use of the product; if applicable, choose the products of corresponding models according to the patient's situation, and prepare the products of different models and specifications before the surgery.
- d) Unless design allowance or clear instructions in the technical manual, the implant cannot be processed and/or changed in any way, including correction, bending, scratching on the shape of the products, all these will make it failure.
- e) Doctors should make the patients fully understand the products and the risks and limitations of

surgery as well as the importance of compliance with doctor orders


- f) Doctors should be familiar with implantation technique and use surgical instruments skillfully.
- g) Scratches on implant surface during surgery will affect its service life, delayed fracture healing or nonunion will lead to implants loosening, bending or breaking.
- h) Under the guidance of the doctor, the patient can do slight flexion-extension movement with the aid of the stick at least a month after operation, and the large amount of exercise or weight-bearing exercise is prohibited. After four months the patient can do normal activities under the guidance of doctor. That taking the activities against the requirements of doctors and weight-bearing activities will result in implants shifting or breaking.
- i) The removal time of implants should be decided by the rehabilitation of the patients after operation, generally about 12 months after operation.
- j) X-ray examination after operation should be done regularly.

9. Warnings

- a) **This product is an implant for single use, which can not be re-used.**
- b) **Metallic bone plates must be used with screws of same series produced by the company, the material should remain consistency.**
- c) **Total weight bearing at wound site is forbidden before fracture healing.**

10. Packaging Identification.

10.1 Product identification

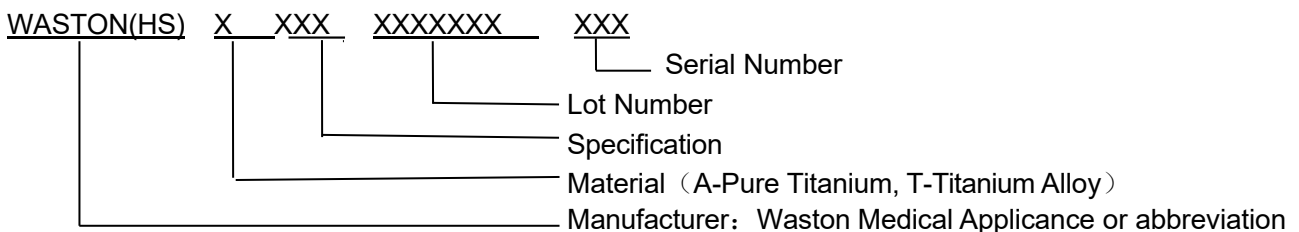

10.2 The packaging shall provide at least the following information:

- a) Manufacturer name , address , trademark
- b) European representatives name, address
- c) CE identification and code of Notify Body
- d) Material name or represent
- e) Product name , model and specification
- f) Sterilization valid period
- g) «Do not reuse» should be marked
- h) Lot number and date of production
- i) Max. storage temperature 40°
- j) Min and max humidity 30-80%
- k) Sterilization method

11. Application methods

According to the fracture site and soft tissue situation determined by pre-operative X-ray, the doctor shall draw up detailed preoperative plan for fracture reduction, choosing correctly bone plates, screws and operation position, installing bone plates and screws right after the line drilling, reaming, and tapping. The entire surgery process must follow The “AO Principles of Fracture Treatment” and refer to the relevant chapters of Orthopedic Surgery.

12. Storage Condition

The scratching, collision of implants in the processes of transportation and usage will undermine the strength and anti-fatigue properties of implants, therefore the proper custody and use of implants is necessary, otherwise, there will be there will be ignorable risks.

Implants should be stored in a well-ventilated room with no more than 80% relative humidity, non-corrosive gases.

13. Service life

Internal fixation is mainly responsible for the physiologic loading before bone healing, and generally it removed after a year when bone healing, otherwise, the risks of re-fracture and implants fracture still exists. The patient should follow the doctor's orders and be followed-up on time. It is a normal phenomenon that implants fracture caused by the removal out of body not promptly after bone healing.

14. Revision Date

April 26, 2015

Annex I: Product list and specifications

REF Number	Product Description
30722-005	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 5 Holes
30722-007	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 7 Holes
30722-009	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 9 Holes
30722-011	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 11 Holes
30722-013	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 13 Holes
30722-105	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 5 Holes
30722-107	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 7 Holes
30722-109	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 9 Holes
30722-111	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 11 Holes
30722-113	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 13 Holes
30722-405	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 5 Holes
30722-407	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 7 Holes
30722-409	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 9 Holes
30722-411	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 11 Holes
30722-413	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 13 Holes
30722-505	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 5 Holes
30722-507	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 7 Holes
30722-509	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 9 Holes
30722-511	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 11 Holes
30722-513	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 13 Holes
30719-003	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes
30719-005	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes
30719-007	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes
30719-009	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 9 Holes
30719-011	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 11 Holes
30719-013	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 13 Holes
30719-103	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes
30719-105	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes
30719-107	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes
30719-109	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 9 Holes
30719-111	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 11 Holes
30719-113	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 13 Holes
30719-203	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 3 Holes
30719-205	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 5 Holes
30719-207	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 7 Holes
30719-209	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 9 Holes
30719-211	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 11 Holes
30719-213	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 13 Holes
30719-303	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 3 Holes
30719-305	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 5 Holes
30719-307	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 7 Holes
30719-309	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 9 Holes
30719-311	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 11 Holes
30719-313	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 13 Holes
30718-003	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes
30718-005	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes
30718-007	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes
30718-009	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 9 Holes
30718-011	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 11 Holes
30718-013	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 13 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30718-103	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes
30718-105	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes
30718-107	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes
30718-109	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 9 Holes
30718-111	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 11 Holes
30718-113	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 13 Holes
10725-011	Distal Lateral Tibial Locking Plate II 5.0, left, 11 Holes
10725-111	Distal Lateral Tibial Locking Plate II 5.0, right, 11 Holes
10725-005	Distal Lateral Tibial Locking Plate II 5.0, left, 5 Holes
10725-105	Distal Lateral Tibial Locking Plate II 5.0, right, 5 Holes
10725-007	Distal Lateral Tibial Locking Plate II 5.0, left, 7 Holes
10725-107	Distal Lateral Tibial Locking Plate II 5.0, right, 7 Holes
10725-009	Distal Lateral Tibial Locking Plate II 5.0, left, 9 Holes
10725-109	Distal Lateral Tibial Locking Plate II 5.0, right, 9 Holes
10737-005	Distal Lateral Tibial Locking Plate II 3.5, left, 5 Holes
10737-007	Distal Lateral Tibial Locking Plate II 3.5, left, 7 Holes
10737-009	Distal Lateral Tibial Locking Plate II 3.5, left, 9 Holes
10737-011	Distal Lateral Tibial Locking Plate II 3.5, left, 11 Holes
10737-013	Distal Lateral Tibial Locking Plate II 3.5, left, 13 Holes
10737-105	Distal Lateral Tibial Locking Plate II 3.5, right, 5 Holes
10737-107	Distal Lateral Tibial Locking Plate II 3.5, right, 7 Holes
10737-109	Distal Lateral Tibial Locking Plate II 3.5, right, 9 Holes
10737-111	Distal Lateral Tibial Locking Plate II 3.5, right, 11 Holes
10737-113	Distal Lateral Tibial Locking Plate II 3.5, right, 13 Holes
10725-203	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 3 Holes
10725-205	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes
10725-207	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 7 Holes
10725-303	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 3 Holes
10725-305	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes
10725-307	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 7 Holes
10709-003	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 3 Holes
10709-004	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 4 Holes
10709-005	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 5 Holes
10709-007	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 7 Holes
10709-103	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 3 Holes
10709-104	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 4 Holes
10709-105	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 5 Holes
10709-107	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 7 Holes
30724-004	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 4 Holes
30724-006	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 6 Holes
30724-008	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 8 Holes
30724-010	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 10 Holes
30724-012	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 12 Holes
30724-104	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 4 Holes
30724-106	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 6 Holes
30724-108	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 8 Holes
30724-110	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 10 Holes
30724-112	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 12 Holes
10725-405	Distal Tibial Locking Plate 3.5, Twin Hole, left, 5 Holes
10725-407	Distal Tibial Locking Plate 3.5, Twin Hole, left, 7 Holes
10725-505	Distal Tibial Locking Plate 3.5, Twin Hole, right, 5 Holes
10725-507	Distal Tibial Locking Plate 3.5, Twin Hole, right, 7 Holes
11042-007	Distal Ulnar Hook Locking Plate II 2.0, Twin Hole, 7 Holes
11042-008	Distal Ulnar Hook Locking Plate II 2.0, Twin Hole, 8 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

10748-006	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 6 Holes
10748-008	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 8 Holes
10748-010	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 10 Holes
10748-112	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 12 Holes
10747-002	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 2 Holes
10747-003	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 3 Holes
10747-004	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 4 Holes
10747-005	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 5 Holes
10747-006	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 6 Holes
10747-008	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 8 Holes
10747-010	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 10 Holes
10747-012	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 12 Holes
10742-005	Femoral Neck Locking Plate 4.5/5.0/6.5, left, 5 Holes
10742-105	Femoral Neck Locking Plate 4.5/5.0/6.5, right, 5 Holes
10741-005	Femoral Trochanter Locking Plate 4.5/5.0/6.5, left, 5 Holes
10741-105	Femoral Trochanter Locking Plate 4.5/5.0/6.5, right, 5 Holes
30717-103	Olecranon Locking Plate III 3.5, Twin Hole, left, 3 Holes
30717-105	Olecranon Locking Plate III 3.5, Twin Hole, left, 5 Holes
30717-107	Olecranon Locking Plate III 3.5, Twin Hole, left, 7 Holes
30717-109	Olecranon Locking Plate III 3.5, Twin Hole, left, 9 Holes
30717-111	Olecranon Locking Plate III 3.5, Twin Hole, left, 11 Holes
30717-113	Olecranon Locking Plate III 3.5, Twin Hole, left, 13 Holes
30717-203	Olecranon Locking Plate III 3.5, Twin Hole, right, 3 Holes
30717-205	Olecranon Locking Plate III 3.5, Twin Hole, right, 5 Holes
30717-207	Olecranon Locking Plate III 3.5, Twin Hole, right, 7 Holes
30717-209	Olecranon Locking Plate III 3.5, Twin Hole, right, 9 Holes
30717-211	Olecranon Locking Plate III 3.5, Twin Hole, right, 11 Holes
30717-213	Olecranon Locking Plate III 3.5, Twin Hole, right, 13 Holes
30749-005	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 5 Holes
30749-006	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 6 Holes
30749-008	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 8 Holes
30749-010	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 10 Holes
30749-012	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 12 Holes
30749-014	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 14 Holes
30716-005	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 5 Holes
30716-007	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 7 Holes
30716-009	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 9 Holes
30716-011	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 11 Holes
30716-013	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 13 Holes
30716-105	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 5 Holes
30716-107	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 7 Holes
30716-109	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 9 Holes
30716-111	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 11 Holes
30716-113	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 13 Holes
30723-003	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 3 Holes
30723-005	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 5 Holes
30723-007	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 7 Holes
30723-009	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 9 Holes
30723-011	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 11 Holes
30723-013	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 13 Holes
30723-103	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 3 Holes
30723-105	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 5 Holes
30723-107	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 7 Holes
30723-109	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 9 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30723-111	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 11 Holes
30723-113	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 13 Holes
30723-205	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 5 Holes
30723-207	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 6 Holes
30723-209	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 9 Holes
30723-211	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 11 Holes
30723-213	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 13 Holes
30723-305	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 5 Holes
30723-307	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 6 Holes
30723-309	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 9 Holes
30723-311	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 11 Holes
30723-313	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 13 Holes
30727-404	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 4 Holes
30727-405	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes
30727-406	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 6 Holes
30727-408	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 8 Holes
30727-410	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 10 Holes
30727-412	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 12 Holes
30727-504	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 4 Holes
30727-505	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes
30727-506	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 6 Holes
30727-508	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 8 Holes
30727-510	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 10 Holes
30727-512	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 12 Holes
30727-604	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 4 Holes
30727-605	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 5 Holes
30727-606	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 6 Holes
30727-608	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 8 Holes
30727-610	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 10 Holes
30727-612	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 12 Holes
30727-704	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 4 Holes
30727-705	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 5 Holes
30727-706	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 6 Holes
30727-708	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 8 Holes
30727-710	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 10 Holes
30727-712	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 12 Holes
30726-204	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 4 Holes
30726-205	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes
30726-206	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 6 Holes
30726-208	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 8 Holes
30726-210	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 10 Holes
30726-212	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 12 Holes
30726-304	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 4 Holes
30726-305	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes
30726-306	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 6 Holes
30726-308	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 8 Holes
30726-310	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 10 Holes
30726-312	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 12 Holes
30726-404	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 4 Holes
30726-504	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 4 Holes
30726-405	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 5 Holes
30726-505	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 5 Holes
30726-406	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 6 Holes
30726-506	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 6 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30726-408	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 8 Holes
30726-508	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 8 Holes
30726-410	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 10 Holes
30726-510	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 10 Holes
30726-412	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 12 Holes
30726-512	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 12 Holes
10730-057	Calcaneal Locking Plate II 3.5, left, 15 Holes, length 57mm
10730-067	Calcaneal Locking Plate II 3.5, left, 15 Holes, length 67mm
10730-157	Calcaneal Locking Plate II 3.5, right, 15 Holes, length 57mm
10730-167	Calcaneal Locking Plate II 3.5, right, 15 Holes, length 67mm
10734-004	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 4 Holes
10734-005	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 5 Holes
10734-006	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 6 Holes
10734-007	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 7 Holes
10734-104	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 4 Holes
10734-105	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 5 Holes
10734-106	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 6 Holes
10734-107	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 7 Holes
10734-204	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 4 Holes
10734-205	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 5 Holes
10734-206	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 6 Holes
10734-207	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 7 Holes
10734-304	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 4 Holes
10734-305	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 5 Holes
10734-306	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 6 Holes
10734-307	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 7 Holes
10734-404	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 4 Holes
10734-405	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 5 Holes
10734-406	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 6 Holes
10734-407	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 7 Holes
10734-504	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 4 Holes
10734-505	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 5 Holes
10734-506	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 6 Holes
10734-507	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 7 Holes
10732-010	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 10 Holes
10732-006	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 6 Holes
10732-007	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 7 Holes
10732-008	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 8 Holes
10732-110	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 10 Holes
10732-106	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 6 Holes
10732-107	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 7 Holes
10732-108	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 8 Holes
10403-050	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 50mm
10403-055	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 55mm
10403-060	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 60mm
10403-065	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 65mm
10403-070	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 70mm
10403-075	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 75mm
10403-080	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 80mm
10403-085	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 85mm
10403-090	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 90mm
10403-095	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 95mm
10403-100	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 100mm
10403-105	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 105mm

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

10403-110	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 110mm
10733-003	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes
10733-004	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 4 Holes
10733-005	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes
10733-006	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 6 Holes
10733-007	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes
10733-008	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, left, 8 Holes
10733-103	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes
10733-104	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 4 Holes
10733-105	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes
10733-106	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 6 Holes
10733-107	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes
10733-108	Distal Clavicula Locking Plate III 2.7/3.5, Twin Hole, right, 8 Holes
30740-011	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 11 Holes
30740-013	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 13 Holes
30740-003	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 3 Holes
30740-005	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 5 Holes
30740-007	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 7 Holes
30740-009	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 9 Holes
10905-203	Distal Dorsal Radial Locking L-Plate 2.4/2.7, left, 3 Holes
10905-204	Distal Dorsal Radial Locking L-Plate 2.4/2.7, left, 4 Holes
10905-303	Distal Dorsal Radial Locking L-Plate 2.4/2.7, right, 3 Holes
10905-304	Distal Dorsal Radial Locking L-Plate 2.4/2.7, right, 4 Holes
10905-504	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, right, 4 Holes
10905-403	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, left, 3 Holes
10905-404	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, left, 4 Holes
10905-503	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, right, 3 Holes
10905-003	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, left, 3 Holes
10905-004	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, left, 4 Holes
10905-103	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, right, 3 Holes
10905-104	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, right, 4 Holes
10906-005	Distal Dorsal Radial Locking Plate 2.4/2.7, Twin Hole, 5 Holes
10906-006	Distal Dorsal Radial Locking Plate 2.4/2.7, Twin Hole, 6 Holes
10904-003	Distal Dorsal Radial Locking T-Plate 2.4/2.7 Twin Hole, 3 Holes
10904-004	Distal Dorsal Radial Locking T-Plate 2.4/2.7 Twin Hole, 4 Holes
30714-011	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 11 Holes
30714-013	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 13 Holes
30714-003	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 3 Holes
30714-005	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 5 Holes
30714-007	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 7 Holes
30714-009	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 9 Holes
30714-111	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 11 Holes
30714-113	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 13 Holes
30714-103	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 3 Holes
30714-105	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 5 Holes
30714-107	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 7 Holes
30714-109	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 9 Holes
10715-010	Distal Lateral Fibula Locking Plate II 3.5 left, 10 Holes
10715-012	Distal Lateral Fibula Locking Plate II 3.5, left, 12 Holes
10715-004	Distal Lateral Fibula Locking Plate II 3.5, left, 4 Holes
10715-006	Distal Lateral Fibula Locking Plate II 3.5, left, 6 Holes
10715-008	Distal Lateral Fibula Locking Plate II 3.5, left, 8 Holes
10715-110	Distal Lateral Fibula Locking Plate II 3.5, right, 10 Holes
10715-112	Distal Lateral Fibula Locking Plate II 3.5, right, 12 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

10715-104	Distal Lateral Fibula Locking Plate II 3.5, right, 4 Holes
10715-106	Distal Lateral Fibula Locking Plate II 3.5, right, 6 Holes
10715-108	Distal Lateral Fibula Locking Plate II 3.5, right, 8 Holes
10709-403	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 3 Holes
10709-404	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 4 Holes
10709-405	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 5 Holes
10709-407	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 7 Holes
10709-503	Distal Medial Radial Locking Plate IV 2.4/3.5, right, 3 Holes
10709-504	Distal Medial Radial Locking Plate IIV 2.4/3.5, right, 4 Holes
10709-505	Distal Medial Radial Locking Plate III 2.4/3.5, right, 5 Holes
10709-507	Distal Medial Radial Locking Plate III 2.4/3.5, right, 7 Holes
10901-003	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 3 Holes
10901-004	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 4 Holes
10901-005	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 5 Holes
10901-103	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 3 Holes
10901-104	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 4 Holes
10901-105	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 5 Holes
30731-203	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 3 Holes
30731-204	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 4 Holes
30731-205	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 5 Holes
30731-206	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 6 Holes
30731-207	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 7 Holes
30731-208	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 8 Holes
30731-303	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 3 Holes
30731-304	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 4 Holes
30731-305	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 5 Holes
30731-306	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 6 Holes
30731-307	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 7 Holes
30731-308	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 8 Holes
30731-403	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 3 Holes
30731-404	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 4 Holes
30731-405	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 5 Holes
30731-406	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 6 Holes
30731-407	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 7 Holes
30731-408	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 8 Holes
30731-503	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 3 Holes
30731-504	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 4 Holes
30731-505	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 5 Holes
30731-506	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 6 Holes
30731-507	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 7 Holes
30731-508	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 8 Holes
30731-003	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 3 Holes
30731-004	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 4 Holes
30731-005	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 5 Holes
30731-006	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 6 Holes
30731-007	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 7 Holes
30731-008	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 8 Holes
30731-103	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 3 Holes
30731-104	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 4 Holes
30731-105	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 5 Holes
30731-106	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 6 Holes
30731-107	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 7 Holes
30731-108	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 8 Holes
30748-004	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 4 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30748-104	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 4 Holes
30748-006	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 6 Holes
30748-106	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 6 Holes
30748-008	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 8 Holes
30748-108	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 8 Holes
30748-010	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 10 Holes
30748-110	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 10 Holes
30748-012	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 12 Holes
30748-112	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 12 Holes
30748-014	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 14 Holes
30748-114	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 14 Holes
10907-010	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 10 Holes
10907-012	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 12 Holes
10907-008	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 8 Holes
10903-003	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 3 Holes
10903-004	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 4 Holes
10903-005	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 5 Holes
10903-103	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 3 Holes
10903-104	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 4 Holes
10903-105	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 5 Holes
10902-003	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, left, 3 Holes
10902-005	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, left, 5 Holes
10902-103	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, right, 3 Holes
10902-105	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, right, 5 Holes
10902-203	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, left, 3 Holes
10902-205	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, left, 5 Holes
10902-303	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, right, 3 Holes
10902-305	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, right, 5 Holes
10731-307	Fast Mini Locking Plate 2.4, flex L-Plate, left
10731-207	Fast Mini Locking Plate 2.4, flex, L-Plate, right
10731-007	Fast Mini Locking Plate 2.4, flex, straight
10731-107	Fast Mini Locking Plate 2.4, flex, Y-Plate
10731-208	Fast Mini Locking Plate 2.4, strength, L-Plate, left
10731-308	Fast Mini Locking Plate 2.4, strength, L-Plate, right
10731-008	Fast Mini Locking Plate 2.4, strength, straight
10731-108	Fast Mini Locking Plate 2.4, strength, Y-Plate
30703-006	Large Locking Straight Plate 4.5/5.0, Twin Hole, 6 Holes
30703-007	Large Locking Straight Plate 4.5/5.0, Twin Hole, 7 Holes
30703-008	Large Locking Straight Plate 4.5/5.0, Twin Hole, 8 Holes
30703-009	Large Locking Straight Plate 4.5/5.0, Twin Hole, 9 Holes
30703-010	Large Locking Straight Plate 4.5/5.0, Twin Hole, 10 Holes
30703-012	Large Locking Straight Plate 4.5/5.0, Twin Hole, 12 Holes
30703-014	Large Locking Straight Plate 4.5/5.0, Twin Hole, 14 Holes
30703-016	Large Locking Straight Plate 4.5/5.0, Twin Hole, 16 Holes
30702-006	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 6 Holes
30702-007	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 7 Holes
30702-008	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 8 Holes
30702-009	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 9 Holes
30702-010	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 10 Holes
30702-012	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 12 Holes
30702-014	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 14 Holes
30702-016	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 16 Holes
30729-004	One-third Tubular Locking Plate 3.5, Twin Hole, 4 Holes
30729-005	One-third Tubular Locking Plate 3.5, Twin Hole, 5 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30729-006	One-third Tubular Locking Plate 3.5, Twin Hole, 6 Holes
30729-007	One-third Tubular Locking Plate 3.5, Twin Hole, 7 Holes
30729-008	One-third Tubular Locking Plate 3.5, Twin Hole, 8 Holes
30729-009	One-third Tubular Locking Plate 3.5, Twin Hole, 9 Holes
30729-010	One-third Tubular Locking Plate 3.5, Twin Hole, 10 Holes
30729-012	One-third Tubular Locking Plate 3.5, Twin Hole, 12 Holes
10211-004	T-buttress Plate, left, 4 holes
10211-104	T-buttress Plate, right, 4 holes
10211-005	T-buttress Plate, left, 5 Holes
10211-105	T-buttress Plate, right, 5 Holes
10211-006	T-buttress Plate, left, 6 Holes
10211-106	T-buttress Plate, right, 6 Holes
10211-007	T-buttress Plate, left, 7 Holes
10211-107	T-buttress Plate, right, 7 Holes
10211-008	T-buttress Plate, left, 8 Holes
10211-108	T-buttress Plate, right, 8 Holes
10212-004	L-buttress Plate, left, 4 holes
10212-104	L-buttress Plate, right, 4 holes
10212-005	L-buttress Plate, left, 5 Holes
10212-105	L-buttress Plate, right, 5 Holes
10212-006	L-buttress Plate, left, 6 Holes
10212-106	L-buttress Plate, right, 6 Holes
10212-007	L-buttress Plate, left, 7 Holes
10212-107	L-buttress Plate, right, 7 Holes
10212-008	L-buttress Plate, left, 8 Holes
10212-108	L-buttress Plate, right, 8 Holes
10207-005	Reconstruction Plate, 5 Holes
10207-006	Reconstruction Plate, 6 Holes
10207-007	Reconstruction Plate, 7 Holes
10207-008	Reconstruction Plate, 8 Holes
10207-009	Reconstruction Plate, 9 Holes
10207-010	Reconstruction Plate, 10 Holes
10207-011	Reconstruction Plate, 11 Holes
10207-012	Reconstruction Plate, 12 Holes
10207-013	Reconstruction Plate, 13 Holes
10208-006	Curved Reconstruction Plate, 6 Holes
10208-008	Curved Reconstruction Plate, 8 Holes
10208-010	Curved Reconstruction Plate, 10 Holes
10208-012	Curved Reconstruction Plate, 12 Holes
10208-014	Curved Reconstruction Plate, 14 Holes
10208-016	Curved Reconstruction Plate, 16 Holes
10208-018	Curved Reconstruction Plate, 18 Holes
10743-004	Pelvic Reconstruction Locking Plate 3.5/4.0, 4 Holes
10743-006	Pelvic Reconstruction Locking Plate 3.5/4.0, 6 Holes
10743-008	Pelvic Reconstruction Locking Plate 3.5/4.0, 8 Holes
10743-010	Pelvic Reconstruction Locking Plate 3.5/4.0, 10 Holes
10743-012	Pelvic Reconstruction Locking Plate 3.5/4.0, 12 Holes
10743-014	Pelvic Reconstruction Locking Plate 3.5/4.0, 14 Holes
30715-010	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 10 Holes
30715-012	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 12 Holes
30715-014	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 14 Holes
30715-006	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 6 Holes
30715-008	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 8 Holes
30715-110	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 10 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30715-112	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 12 Holes
30715-114	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 14 Holes
30715-106	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 6 Holes
30715-108	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 8 Holes
30735-003	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 3 Holes
30735-004	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 4 Holes
30735-005	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 5 Holes
30735-006	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 6 Holes
30735-008	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 8 Holes
30735-010	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 10 Holes
30735-012	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 12 Holes
30721-005	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 5 Holes
30721-007	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 7 Holes
30721-009	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 9 Holes
30721-011	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 11 Holes
30721-013	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 13 Holes
30721-105	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 5 Holes
30721-107	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 7 Holes
30721-109	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 9 Holes
30721-111	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 11 Holes
30721-113	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 13 Holes
10739-210	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 10 Holes
10739-212	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 12 Holes
10739-204	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 4 Holes
10739-206	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 6 Holes
10739-208	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 8 Holes
10739-310	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 10 Holes
10739-312	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 12 Holes
10739-304	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 4 Holes
10739-306	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 6 Holes
10739-308	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 8 Holes
30713-003	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 3 Holes
30713-004	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 4 Holes
30713-005	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 5 Holes
30713-006	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 6 Holes
30713-103	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 3 Holes
30713-104	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 4 Holes
30713-105	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 5 Holes
30713-106	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 6 Holes
10704-004	Reconstruction Locking Plate II 3.5, 4 Holes
10704-005	Reconstruction Locking Plate II 3.5, 5 Holes
10704-006	Reconstruction Locking Plate II 3.5, 6 Holes
10704-007	Reconstruction Locking Plate II 3.5, 7 Holes
10704-008	Reconstruction Locking Plate II 3.5, 8 Holes
10704-009	Reconstruction Locking Plate II 3.5, 9 Holes
10704-010	Reconstruction Locking Plate II 3.5, 10 Holes
10704-011	Reconstruction Locking Plate II 3.5, 11 Holes
10704-012	Reconstruction Locking Plate II 3.5, 12 Holes
10704-013	Reconstruction Locking Plate II 3.5, 13 Holes
10704-014	Reconstruction Locking Plate II 3.5, 14 Holes
10746-006	Sacro-Iliac Locking Butterfly-Plate 3.5/4.0, Twin Hole, 6 Holes
30700-004	Small Straight Locking Plate 3.5, Twin Hole, 4 Holes
30700-005	Small Straight Locking Plate 3.5, Twin Hole, 5 Holes
30700-006	Small Straight Locking Plate 3.5, Twin Hole, 6 Holes

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

30700-007	Small Straight Locking Plate 3.5, Twin Hole, 7 Holes
30700-008	Small Straight Locking Plate 3.5, Twin Hole, 8 Holes
30700-009	Small Straight Locking Plate 3.5, Twin Hole, 9 Holes
30700-010	Small Straight Locking Plate 3.5, Twin Hole, 10 Holes
30700-011	Small Straight Locking Plate 3.5, Twin Hole, 11 Holes
30700-012	Small Straight Locking Plate 3.5, Twin Hole, 12 Holes
30700-014	Small Straight Locking Plate 3.5, Twin Hole, 14 Holes
30701-004	Small Straight Locking Plate 3.5, Twin Hole, narrow, 4 Holes
30701-005	Small Straight Locking Plate 3.5, Twin Hole, narrow, 5 Holes
30701-006	Small Straight Locking Plate 3.5, Twin Hole, narrow, 6 Holes
30701-007	Small Straight Locking Plate 3.5, Twin Hole, narrow, 7 Holes
30701-008	Small Straight Locking Plate 3.5, Twin Hole, narrow, 8 Holes
30701-009	Small Straight Locking Plate 3.5, Twin Hole, narrow, 9 Holes
30701-010	Small Straight Locking Plate 3.5, Twin Hole, narrow, 10 Holes
30701-011	Small Straight Locking Plate 3.5, Twin Hole, narrow, 11 Holes
30701-012	Small Straight Locking Plate 3.5, Twin Hole, narrow, 12 Holes
30701-014	Small Straight Locking Plate 3.5, Twin Hole, narrow, 14 Holes
11040-012	Locking Adaption Plate II, 2.0, Twin Hole, 12 Holes
11040-006	Locking Adaption Plate II, 2.0, Twin Hole, 6 Holes
11037-007	Locking Condylar Plate II, 2.0, 7 Holes
11067-007	Locking Condylar Plate II, 2.4 Twin Hole, 7 Holes
11039-007	Locking T-Adaption Plate II, 2.0, Twin Hole, 7 Holes
11025-008	Locking T-Plate II, 1.5, 8 Holes
11068-007	Locking T-Plate II, 2 Head Holes, 2.4, Twin Hole, 7 Holes
11068-107	Locking T-Plate II, 3 Head Holes, 2.4, Twin Hole, 7 Holes
11039-107	Locking T-Plate, II, 2.0, Twin Hole, 7 Holes
11036-007	Locking Y-Adaption Plate II, 2.0, Twin Hole, 7 Holes
11021-008	Locking Y-Plate II, 1.5, 8 Holes
11022-006	Locking Y-Plate III, 1.5, 6 Holes
11038-004	Locking Y-Plate III, 2.0, Twin Hole, 4 Holes
11038-005	Locking Y-Plate III, 2.0, Twin Hole, 5 Holes
11066-007	Locking Y-Adaption Plate II, 2.4, Twin Hole, 7 Holes
11001-012	Reconstruction Plate II 2.0, 12 Holes
11001-006	Reconstruction Plate II 2.0, 6 Holes
11002-004	Rectangular Grille Plate 2.0, 4 Holes
11024-012	Straight Locking Plate II 1.5, Twin Hole, 12 Holes
11024-006	Straight Locking Plate II 1.5, Twin Hole, 6 Holes
11023-004	Straight Locking Plate II, 1.5, Twin Hole, 4 Holes
11023-006	Straight Locking Plate II, 1.5, Twin Hole, 6 Holes
11041-010	Straight Locking Plate II, 2.0, Twin Hole, 10 Holes
11041-004	Straight Locking Plate II, 2.0, Twin Hole, 4 Holes
11041-005	Straight Locking Plate II, 2.0, Twin Hole, 5 Holes
11041-006	Straight Locking Plate II, 2.0, Twin Hole, 6 Holes
11041-007	Straight Locking Plate II, 2.0, Twin Hole, 7 Holes
11041-008	Straight Locking Plate II, 2.0, Twin Hole, 8 Holes
11070-010	Straight Locking Plate II, 2.4, Twin Hole, 10 Holes
11070-004	Straight Locking Plate II, 2.4, Twin Hole, 4 Holes
11070-005	Straight Locking Plate II, 2.4, Twin Hole, 5 Holes
11070-006	Straight Locking Plate II, 2.4, Twin Hole, 6 Holes
11070-007	Straight Locking Plate II, 2.4, Twin Hole, 7 Holes
11070-008	Straight Locking Plate II, 2.4, Twin Hole, 8 Holes
10238-145	Rib Plate , 11x45mm
10238-245	Rib Plate , 15x45mm
10238-255	Rib Plate, 15x55mm

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

10244-055	Rib Plate, 17x45mm
10238-345	Rib Plate, 19x45mm
10238-355	Rib Plate, 19x55mm
10238-845	Rib Plate, 19x45x10mm
10238-855	Rib Plate, 19x55x10mm
10259-355	Rib Plate, 22x55mm
NZB3.B054.00 4	WAVE Proximal Posterior Tibia Locking Plate, left, 4 holes
NZB3.B054.00 6	WAVE Proximal Posterior Tibia Locking Plate, left, 6 holes
NZB3.B055.00 4	WAVE Proximal Posterior Tibia Locking Plate, right, 4 holes
NZB3.B055.00 6	WAVE Proximal Posterior Tibia Locking Plate, right, 6 holes
S30722-005	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 5 Holes (sterile)
S30722-007	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 7 Holes (sterile)
S30722-009	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 9 Holes (sterile)
S30722-011	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 11 Holes (sterile)
S30722-013	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, left, 13 Holes (sterile)
S30722-105	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 5 Holes (sterile)
S30722-107	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 7 Holes (sterile)
S30722-109	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 9 Holes (sterile)
S30722-111	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 11 Holes (sterile)
S30722-113	Distal Lateral Condylar Femoral Locking Plate 4.5/5.0/6.5, Twin Hole, right, 13 Holes (sterile)
S30722-405	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 5 Holes (sterile)
S30722-407	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 7 Holes (sterile)
S30722-409	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 9 Holes (sterile)
S30722-411	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 11 Holes (sterile)
S30722-413	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, left, 13 Holes (sterile)
S30722-505	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 5 Holes (sterile)
S30722-507	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 7 Holes (sterile)
S30722-509	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 9 Holes (sterile)
S30722-511	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 11 Holes (sterile)
S30722-513	Distal Lateral Femoral Locking Plate 4.5/5.0, LISS, Twin Hole, right, 13 Holes (sterile)
S30719-003	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes (sterile)
S30719-005	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes (sterile)
S30719-007	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes (sterile)
S30719-009	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 9 Holes (sterile)
S30719-011	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 11 Holes (sterile)
S30719-013	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 13 Holes (sterile)
S30719-103	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes (sterile)
S30719-105	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes (sterile)
S30719-107	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes (sterile)
S30719-109	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 9 Holes (sterile)
S30719-111	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 11 Holes (sterile)
S30719-113	Distal Lateral Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 13 Holes (sterile)
S30719-203	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 3 Holes (sterile)
S30719-205	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 5 Holes (sterile)
S30719-207	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 7 Holes (sterile)
S30719-209	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 9 Holes (sterile)
S30719-211	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 11 Holes (sterile)
S30719-213	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, left, 13 Holes (sterile)
S30719-303	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 3 Holes (sterile)
S30719-305	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 5 Holes (sterile)
S30719-307	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 7 Holes (sterile)
S30719-309	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 9 Holes (sterile)
S30719-311	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 11 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30719-313	Distal Lateral Humeral Locking Plate IV 2.7/3.5, Twin Hole, right, 13 Holes (sterile)
S30718-003	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes (sterile)
S30718-005	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes (sterile)
S30718-007	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes (sterile)
S30718-009	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 9 Holes (sterile)
S30718-011	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 11 Holes (sterile)
S30718-013	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, left, 13 Holes (sterile)
S30718-103	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes (sterile)
S30718-105	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes (sterile)
S30718-107	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes (sterile)
S30718-109	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 9 Holes (sterile)
S30718-111	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 11 Holes (sterile)
S30718-113	Distal Medial Humeral Locking Plate III 2.7/3.5, Twin Hole, right, 13 Holes (sterile)
S10725-011	Distal Lateral Tibial Locking Plate II 5.0, left, 11 Holes (sterile)
S10725-111	Distal Lateral Tibial Locking Plate II 5.0, right, 11 Holes (sterile)
S10725-005	Distal Lateral Tibial Locking Plate II 5.0, left, 5 Holes (sterile)
S10725-105	Distal Lateral Tibial Locking Plate II 5.0, right, 5 Holes (sterile)
S10725-007	Distal Lateral Tibial Locking Plate II 5.0, left, 7 Holes (sterile)
S10725-107	Distal Lateral Tibial Locking Plate II 5.0, right, 7 Holes (sterile)
S10725-009	Distal Lateral Tibial Locking Plate II 5.0, left, 9 Holes (sterile)
S10725-109	Distal Lateral Tibial Locking Plate II 5.0, right, 9 Holes (sterile)
S10737-005	Distal Lateral Tibial Locking Plate II 3.5, left, 5 Holes (sterile)
S10737-007	Distal Lateral Tibial Locking Plate II 3.5, left, 7 Holes (sterile)
S10737-009	Distal Lateral Tibial Locking Plate II 3.5, left, 9 Holes (sterile)
S10737-011	Distal Lateral Tibial Locking Plate II 3.5, left, 11 Holes (sterile)
S10737-013	Distal Lateral Tibial Locking Plate II 3.5, left, 13 Holes (sterile)
S10737-105	Distal Lateral Tibial Locking Plate II 3.5, right, 5 Holes (sterile)
S10737-107	Distal Lateral Tibial Locking Plate II 3.5, right, 7 Holes (sterile)
S10737-109	Distal Lateral Tibial Locking Plate II 3.5, right, 9 Holes (sterile)
S10737-111	Distal Lateral Tibial Locking Plate II 3.5, right, 11 Holes (sterile)
S10737-113	Distal Lateral Tibial Locking Plate II 3.5, right, 13 Holes (sterile)
S10725-203	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 3 Holes (sterile)
S10725-205	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes (sterile)
S10725-207	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 7 Holes (sterile)
S10725-303	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 3 Holes (sterile)
S10725-305	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes (sterile)
S10725-307	Distal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 7 Holes (sterile)
S10709-003	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 3 Holes (sterile)
S10709-004	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 4 Holes (sterile)
S10709-005	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 5 Holes (sterile)
S10709-007	Distal Volar Medial Radial Locking Plate III 2.4/3.5, left, 7 Holes (sterile)
S10709-103	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 3 Holes (sterile)
S10709-104	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 4 Holes (sterile)
S10709-105	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 5 Holes (sterile)
S10709-107	Distal Volar Medial Radial Locking Plate III 2.4/3.5, right, 7 Holes (sterile)
S30724-004	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 4 Holes (sterile)
S30724-006	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 6 Holes (sterile)
S30724-008	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 8 Holes (sterile)
S30724-010	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 10 Holes (sterile)
S30724-012	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, left, 12 Holes (sterile)
S30724-104	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 4 Holes (sterile)
S30724-106	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 6 Holes (sterile)
S30724-108	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 8 Holes (sterile)
S30724-110	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 10 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30724-112	Distal Medial Tibial Locking Plate II 3.5, Twin Hole, right, 12 Holes (sterile)
S10725-403	Distal Tibial Locking Plate 3.5, Twin Hole, left, 3 Holes (sterile)
S10725-405	Distal Tibial Locking Plate 3.5, Twin Hole, left, 5 Holes (sterile)
S10725-407	Distal Tibial Locking Plate 3.5, Twin Hole, left, 7 Holes (sterile)
S10725-503	Distal Tibial Locking Plate 3.5, Twin Hole, right, 3 Holes (sterile)
S10725-505	Distal Tibial Locking Plate 3.5, Twin Hole, right, 5 Holes (sterile)
S10725-507	Distal Tibial Locking Plate 3.5, Twin Hole, right, 7 Holes (sterile)
S11042-007	Distal Ulnar Hook Locking Plate II 2.0, Twin Hole, 7 Holes (sterile)
S11042-008	Distal Ulnar Hook Locking Plate II 2.0, Twin Hole, 8 Holes (sterile)
S10748-006	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 6 Holes (sterile)
S10748-008	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 8 Holes (sterile)
S10748-010	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 10 Holes (sterile)
S10748-112	Dynamic Condylar Barrel Locking Plate 4.5/5.0, Twin Hole, 12 Holes (sterile)
S10747-002	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 2 Holes (sterile)
S10747-003	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 3 Holes (sterile)
S10747-004	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 4 Holes (sterile)
S10747-005	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 5 Holes (sterile)
S10747-006	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 6 Holes (sterile)
S10747-008	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 8 Holes (sterile)
S10747-010	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 10 Holes (sterile)
S10747-012	Dynamic Hip Screw Locking Plate II 4.5/5.0, Twin Hole, 12 Holes (sterile)
S10742-005	Femoral Neck Locking Plate 4.5/5.0/6.5, left, 5 Holes (sterile)
S10742-105	Femoral Neck Locking Plate 4.5/5.0/6.5, right, 5 Holes (sterile)
S10741-005	Femoral Trochanter Locking Plate 4.5/5.0/6.5, left, 5 Holes (sterile)
S10741-105	Femoral Trochanter Locking Plate 4.5/5.0/6.5, right, 5 Holes (sterile)
S30717-103	Olecranon Locking Plate III 3.5, Twin Hole, left, 3 Holes (sterile)
S30717-105	Olecranon Locking Plate III 3.5, Twin Hole, left, 5 Holes (sterile)
S30717-107	Olecranon Locking Plate III 3.5, Twin Hole, left, 7 Holes (sterile)
S30717-109	Olecranon Locking Plate III 3.5, Twin Hole, left, 9 Holes (sterile)
S30717-111	Olecranon Locking Plate III 3.5, Twin Hole, left, 11 Holes (sterile)
S30717-113	Olecranon Locking Plate III 3.5, Twin Hole, left, 13 Holes (sterile)
S30717-203	Olecranon Locking Plate III 3.5, Twin Hole, right, 3 Holes (sterile)
S30717-205	Olecranon Locking Plate III 3.5, Twin Hole, right, 5 Holes (sterile)
S30717-207	Olecranon Locking Plate III 3.5, Twin Hole, right, 7 Holes (sterile)
S30717-209	Olecranon Locking Plate III 3.5, Twin Hole, right, 9 Holes (sterile)
S30717-211	Olecranon Locking Plate III 3.5, Twin Hole, right, 11 Holes (sterile)
S30717-213	Olecranon Locking Plate III 3.5, Twin Hole, right, 13 Holes (sterile)
S30749-005	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 5 Holes (sterile)
S30749-006	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 6 Holes (sterile)
S30749-008	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 8 Holes (sterile)
S30749-010	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 10 Holes (sterile)
S30749-012	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 12 Holes (sterile)
S30749-014	Posterior Medial Proximal Tibial Plate 3.5, Twin Hole, 14 Holes (sterile)
S30716-005	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 5 Holes (sterile)
S30716-007	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 7 Holes (sterile)
S30716-009	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 9 Holes (sterile)
S30716-011	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 11 Holes (sterile)
S30716-013	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, left, 13 Holes (sterile)
S30716-105	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 5 Holes (sterile)
S30716-107	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 7 Holes (sterile)
S30716-109	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 9 Holes (sterile)
S30716-111	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 11 Holes (sterile)
S30716-113	Proximal Condylar Lateral Femoral Locking Plate II 4.5/5.0/6.5, Twin Hole, right, 13 Holes (sterile)
S30723-003	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 3 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30723-005	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 5 Holes (sterile)
S30723-007	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 7 Holes (sterile)
S30723-009	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 9 Holes (sterile)
S30723-011	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 11 Holes (sterile)
S30723-013	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, left, 13 Holes (sterile)
S30723-103	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 3 Holes (sterile)
S30723-105	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 5 Holes (sterile)
S30723-107	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 7 Holes (sterile)
S30723-109	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 9 Holes (sterile)
S30723-111	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 11 Holes (sterile)
S30723-113	Lateral Tibial Head Buttress Locking Plate III 4.5/5.0, Twin Hole, right, 13 Holes (sterile)
S30723-205	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 5 Holes (sterile)
S30723-207	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 6 Holes (sterile)
S30723-209	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 9 Holes (sterile)
S30723-211	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 11 Holes (sterile)
S30723-213	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, left, 13 Holes (sterile)
S30723-305	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 5 Holes (sterile)
S30723-307	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 6 Holes (sterile)
S30723-309	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 9 Holes (sterile)
S30723-311	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 11 Holes (sterile)
S30723-313	Proximal Lateral Tibial Locking Plate 4.5/5.0, LISS, Twin Hole, right, 13 Holes (sterile)
S30727-404	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 4 Holes (sterile)
S30727-405	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes (sterile)
S30727-406	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 6 Holes (sterile)
S30727-408	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 8 Holes (sterile)
S30727-410	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 10 Holes (sterile)
S30727-412	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, left, 12 Holes (sterile)
S30727-504	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 4 Holes (sterile)
S30727-505	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes (sterile)
S30727-506	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 6 Holes (sterile)
S30727-508	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 8 Holes (sterile)
S30727-510	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 10 Holes (sterile)
S30727-512	Proximal Lateral Tibial Locking Plate III 3.5, Twin Hole, right, 12 Holes (sterile)
S30727-604	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 4 Holes (sterile)
S30727-605	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 5 Holes (sterile)
S30727-606	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 6 Holes (sterile)
S30727-608	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 8 Holes (sterile)
S30727-610	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 10 Holes (sterile)
S30727-612	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, left, 12 Holes (sterile)
S30727-704	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 4 Holes (sterile)
S30727-705	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 5 Holes (sterile)
S30727-706	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 6 Holes (sterile)
S30727-708	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 8 Holes (sterile)
S30727-710	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 10 Holes (sterile)
S30727-712	Proximal Lateral Tibial Locking Plate IV 4.5/5.0/6.5, Twin Hole, right, 12 Holes (sterile)
S30726-204	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 4 Holes (sterile)
S30726-205	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 5 Holes (sterile)
S30726-206	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 6 Holes (sterile)
S30726-208	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 8 Holes (sterile)
S30726-210	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 10 Holes (sterile)
S30726-212	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, left, 12 Holes (sterile)
S30726-304	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 4 Holes (sterile)
S30726-305	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 5 Holes (sterile)
S30726-306	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 6 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30726-308	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 8 Holes (sterile)
S30726-310	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 10 Holes (sterile)
S30726-312	Proximal Medial Tibial Locking Plate III 3.5, Twin Hole, right, 12 Holes (sterile)
S30726-404	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 4 Holes (sterile)
S30726-504	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 4 Holes (sterile)
S30726-405	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 5 Holes (sterile)
S30726-505	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 5 Holes (sterile)
S30726-406	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 6 Holes (sterile)
S30726-506	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 6 Holes (sterile)
S30726-408	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 8 Holes (sterile)
S30726-508	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 8 Holes (sterile)
S30726-410	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 10 Holes (sterile)
S30726-510	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 10 Holes (sterile)
S30726-412	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, left, 12 Holes (sterile)
S30726-512	Proximal Medial Tibial Locking Plate IV 4.5/5.0, Twin Hole, right, 12 Holes (sterile)
S10730-057	Calcaneal Locking Plate II 3.5, left, 15 Holes, length 57mm (sterile)
S10730-067	Calcaneal Locking Plate II 3.5, left, 15 Holes, length 67mm (sterile)
S10730-157	Calcaneal Locking Plate II 3.5, right, 15 Holes, length 57mm (sterile)
S10730-167	Calcaneal Locking Plate II 3.5, right, 15 Holes, length 67mm (sterile)
S10734-004	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 4 Holes (sterile)
S10734-005	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 5 Holes (sterile)
S10734-006	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 6 Holes (sterile)
S10734-007	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, left, 7 Holes (sterile)
S10734-104	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 4 Holes (sterile)
S10734-105	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 5 Holes (sterile)
S10734-106	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 6 Holes (sterile)
S10734-107	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 12mm, right, 7 Holes (sterile)
S10734-204	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 4 Holes (sterile)
S10734-205	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 5 Holes (sterile)
S10734-206	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 6 Holes (sterile)
S10734-207	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, left, 7 Holes (sterile)
S10734-304	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 4 Holes (sterile)
S10734-305	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 5 Holes (sterile)
S10734-306	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 6 Holes (sterile)
S10734-307	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 15mm, right, 7 Holes (sterile)
S10734-404	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 4 Holes (sterile)
S10734-405	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 5 Holes (sterile)
S10734-406	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 6 Holes (sterile)
S10734-407	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, left, 7 Holes (sterile)
S10734-504	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 4 Holes (sterile)
S10734-505	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 5 Holes (sterile)
S10734-506	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 6 Holes (sterile)
S10734-507	Clavicula Hook Plate III 3.5/4.0, Twin Hole, 18mm, right, 7 Holes (sterile)
S10732-010	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 10 Holes (sterile)
S10732-006	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 6 Holes (sterile)
S10732-007	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 7 Holes (sterile)
S10732-008	Clavicula Locking Plate III 3.5/4.0, Twin Hole, left, 8 Holes (sterile)
S10732-110	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 10 Holes (sterile)
S10732-106	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 6 Holes (sterile)
S10732-107	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 7 Holes (sterile)
S10732-108	Clavicula Locking Plate III 3.5/4.0, Twin Hole, right, 8 Holes (sterile)
S10403-050	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 50mm (sterile)
S10403-055	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 55mm (sterile)
S10403-060	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 60mm (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S10403-065	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 65mm (sterile)
S10403-070	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 70mm (sterile)
S10403-075	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 75mm (sterile)
S10403-080	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 80mm (sterile)
S10403-085	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 85mm (sterile)
S10403-090	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 90mm (sterile)
S10403-095	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 95mm (sterile)
S10403-100	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 100mm (sterile)
S10403-105	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 105mm (sterile)
S10403-110	Lag Screw for Dynamic Hip/ Dynamic Condylar Plate II, 110mm (sterile)
S10733-003	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 3 Holes (sterile)
S10733-004	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 4 Holes (sterile)
S10733-005	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 5 Holes (sterile)
S10733-006	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 6 Holes (sterile)
S10733-007	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 7 Holes (sterile)
S10733-008	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, left, 8 Holes (sterile)
S10733-103	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 3 Holes (sterile)
S10733-104	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 4 Holes (sterile)
S10733-105	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 5 Holes (sterile)
S10733-106	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 6 Holes (sterile)
S10733-107	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 7 Holes (sterile)
S10733-108	Distal Clavícula Locking Plate III 2.7/3.5, Twin Hole, right, 8 Holes (sterile)
S30740-011	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 11 Holes (sterile)
S30740-013	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 13 Holes (sterile)
S30740-003	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 3 Holes (sterile)
S30740-004	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 4 Holes (sterile)
S30740-005	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 5 Holes (sterile)
S30740-007	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 7 Holes (sterile)
S30740-009	Distal Dorsal Lateral Fibula Locking Plate 3.5, Twin Hole, 9 Holes (sterile)
S10905-203	Distal Dorsal Radial Locking L-Plate 2.4/2.7, left, 3 Holes (sterile)
S10905-204	Distal Dorsal Radial Locking L-Plate 2.4/2.7, left, 4 Holes (sterile)
S10905-303	Distal Dorsal Radial Locking L-Plate 2.4/2.7, right, 3 Holes (sterile)
S10905-304	Distal Dorsal Radial Locking L-Plate 2.4/2.7, right, 4 Holes (sterile)
S10905-504	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, right, 4 Holes (sterile)
S10905-403	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, left, 3 Holes (sterile)
S10905-404	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, left, 4 Holes (sterile)
S10905-503	Distal Dorsal Radial Locking L-Plate 2.4/2.7, small, Twin Hole, right, 3 Holes (sterile)
S10905-003	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, left, 3 Holes (sterile)
S10905-004	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, left, 4 Holes (sterile)
S10905-103	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, right, 3 Holes (sterile)
S10905-104	Distal Dorsal Radial Locking L-Plate 2.4/2.7, Oblique, Twin Hole, right, 4 Holes (sterile)
S10906-005	Distal Dorsal Radial Locking Plate 2.4/2.7, Twin Hole, 5 Holes (sterile)
S10906-006	Distal Dorsal Radial Locking Plate 2.4/2.7, Twin Hole, 6 Holes (sterile)
S10904-003	Distal Dorsal Radial Locking T-Plate 2.4/2.7 Twin Hole, 3 Holes (sterile)
S10904-004	Distal Dorsal Radial Locking T-Plate 2.4/2.7 Twin Hole, 4 Holes (sterile)
S30714-011	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 11 Holes (sterile)
S30714-013	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 13 Holes (sterile)
S30714-003	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 3 Holes (sterile)
S30714-005	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 5 Holes (sterile)
S30714-007	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 7 Holes (sterile)
S30714-009	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, left, 9 Holes (sterile)
S30714-111	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 11 Holes (sterile)
S30714-113	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 13 Holes (sterile)
S30714-103	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 3 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30714-105	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 5 Holes (sterile)
S30714-107	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 7 Holes (sterile)
S30714-109	Distal Lateral Fibula Locking Compression Plate 2.7/3.5, Twin Hole, right, 9 Holes (sterile)
S10715-010	Distal Lateral Fibula Locking Plate II 3.5 left, 10 Holes (sterile)
S10715-012	Distal Lateral Fibula Locking Plate II 3.5, left, 12 Holes (sterile)
S10715-004	Distal Lateral Fibula Locking Plate II 3.5, left, 4 Holes (sterile)
S10715-006	Distal Lateral Fibula Locking Plate II 3.5, left, 6 Holes (sterile)
S10715-008	Distal Lateral Fibula Locking Plate II 3.5, left, 8 Holes (sterile)
S10715-110	Distal Lateral Fibula Locking Plate II 3.5, right, 10 Holes (sterile)
S10715-112	Distal Lateral Fibula Locking Plate II 3.5, right, 12 Holes (sterile)
S10715-104	Distal Lateral Fibula Locking Plate II 3.5, right, 4 Holes (sterile)
S10715-106	Distal Lateral Fibula Locking Plate II 3.5, right, 6 Holes (sterile)
S10715-108	Distal Lateral Fibula Locking Plate II 3.5, right, 8 Holes (sterile)
S10709-403	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 3 Holes (sterile)
S10709-404	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 4 Holes (sterile)
S10709-405	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 5 Holes (sterile)
S10709-407	Distal Medial Radial Locking Plate IV 2.4/3.5, left, 7 Holes (sterile)
S10709-503	Distal Medial Radial Locking Plate IV 2.4/3.5, right, 3 Holes (sterile)
S10709-504	Distal Medial Radial Locking Plate IIV 2.4/3.5, right, 4 Holes (sterile)
S10709-505	Distal Medial Radial Locking Plate III 2.4/3.5, right, 5 Holes (sterile)
S10709-507	Distal Medial Radial Locking Plate III 2.4/3.5, right, 7 Holes (sterile)
S10901-003	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 3 Holes (sterile)
S10901-004	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 4 Holes (sterile)
S10901-005	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, left, 5 Holes (sterile)
S10901-103	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 3 Holes (sterile)
S10901-104	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 4 Holes (sterile)
S10901-105	Distal Volar Medial Radial Locking Plate III 2.4/2.7, Twin Hole, right, 5 Holes (sterile)
S30731-203	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 3 Holes (sterile)
S30731-204	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 4 Holes (sterile)
S30731-205	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 5 Holes (sterile)
S30731-206	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 6 Holes (sterile)
S30731-207	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 7 Holes (sterile)
S30731-208	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, left, 8 Holes (sterile)
S30731-303	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 3 Holes (sterile)
S30731-304	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 4 Holes (sterile)
S30731-305	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 5 Holes (sterile)
S30731-306	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 6 Holes (sterile)
S30731-307	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 7 Holes (sterile)
S30731-308	Distal Radius Locking Plate III 2.4, Twin Hole, small, W=20mm, right, 8 Holes (sterile)
S30731-403	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 3 Holes (sterile)
S30731-404	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 4 Holes (sterile)
S30731-405	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 5 Holes (sterile)
S30731-406	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 6 Holes (sterile)
S30731-407	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 7 Holes (sterile)
S30731-408	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, left, 8 Holes (sterile)
S30731-503	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 3 Holes (sterile)
S30731-504	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 4 Holes (sterile)
S30731-505	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 5 Holes (sterile)
S30731-506	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 6 Holes (sterile)
S30731-507	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 7 Holes (sterile)
S30731-508	Distal Radius Locking Plate III 2.4, Twin Hole, medium, W=22.5mm, right, 8 Holes (sterile)
S30731-003	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 3 Holes (sterile)
S30731-004	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 4 Holes (sterile)
S30731-005	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 5 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30731-006	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 6 Holes (sterile)
S30731-007	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 7 Holes (sterile)
S30731-008	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, left, 8 Holes (sterile)
S30731-103	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 3 Holes (sterile)
S30731-104	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 4 Holes (sterile)
S30731-105	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 5 Holes (sterile)
S30731-106	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 6 Holes (sterile)
S30731-107	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 7 Holes (sterile)
S30731-108	Distal Radius Locking Plate III 2.4, Twin Hole, large, W=25mm, right, 8 Holes (sterile)
S30748-004	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 4 Holes (sterile)
S30748-104	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 4 Holes (sterile)
S30748-006	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 6 Holes (sterile)
S30748-106	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 6 Holes (sterile)
S30748-008	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 8 Holes (sterile)
S30748-108	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 8 Holes (sterile)
S30748-010	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 10 Holes (sterile)
S30748-110	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 10 Holes (sterile)
S30748-012	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 12 Holes (sterile)
S30748-112	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 12 Holes (sterile)
S30748-014	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, left, 14 Holes (sterile)
S30748-114	Distal Medial Tibial Locking Plate III 3.5/4.0, Twin Hole, right, 14 Holes (sterile)
S10907-010	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 10 Holes (sterile)
S10907-012	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 12 Holes (sterile)
S10907-008	Distal Radial Shaft Locking T-Plate 2.4/2.7, Twin Hole, 8 Holes (sterile)
S10903-003	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 3 Holes (sterile)
S10903-004	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 4 Holes (sterile)
S10903-005	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, left, 5 Holes (sterile)
S10903-103	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 3 Holes (sterile)
S10903-104	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 4 Holes (sterile)
S10903-105	Distal Volar Radial Column Locking Plate 2.4/2.7, Twin Hole, right, 5 Holes (sterile)
S10902-003	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, left, 3 Holes (sterile)
S10902-005	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, left, 5 Holes (sterile)
S10902-103	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, right, 3 Holes (sterile)
S10902-105	Distal Volar Radial Locking Plate, 4 Head holes 2.4/2.7 Twin Hole, right, 5 Holes (sterile)
S10902-203	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, left, 3 Holes (sterile)
S10902-205	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, left, 5 Holes (sterile)
S10902-303	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, right, 3 Holes (sterile)
S10902-305	Distal Volar Radial Locking Plate, 5 Head holes 2.4/2.7 Twin Hole, right, 5 Holes (sterile)
S10731-307	Fast Mini Locking Plate 2.4, flex L-Plate, left (sterile)
S10731-207	Fast Mini Locking Plate 2.4, flex, L-Plate, right (sterile)
S10731-007	Fast Mini Locking Plate 2.4, flex, straight (sterile)
S10731-107	Fast Mini Locking Plate 2.4, flex, Y-Plate (sterile)
S10731-208	Fast Mini Locking Plate 2.4, strength, L-Plate, left (sterile)
S10731-308	Fast Mini Locking Plate 2.4, strength, L-Plate, right (sterile)
S10731-008	Fast Mini Locking Plate 2.4, strength, straight (sterile)
S10731-108	Fast Mini Locking Plate 2.4, strength, Y-Plate (sterile)
S30703-006	Large Locking Straight Plate 4.5/5.0, Twin Hole, 6 Holes (sterile)
S30703-007	Large Locking Straight Plate 4.5/5.0, Twin Hole, 7 Holes (sterile)
S30703-008	Large Locking Straight Plate 4.5/5.0, Twin Hole, 8 Holes (sterile)
S30703-009	Large Locking Straight Plate 4.5/5.0, Twin Hole, 9 Holes (sterile)
S30703-010	Large Locking Straight Plate 4.5/5.0, Twin Hole, 10 Holes (sterile)
S30703-012	Large Locking Straight Plate 4.5/5.0, Twin Hole, 12 Holes (sterile)
S30703-014	Large Locking Straight Plate 4.5/5.0, Twin Hole, 14 Holes (sterile)
S30703-016	Large Locking Straight Plate 4.5/5.0, Twin Hole, 16 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S30702-006	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 6 Holes (sterile)
S30702-007	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 7 Holes (sterile)
S30702-008	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 8 Holes (sterile)
S30702-009	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 9 Holes (sterile)
S30702-010	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 10 Holes (sterile)
S30702-012	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 12 Holes (sterile)
S30702-014	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 14 Holes (sterile)
S30702-016	Large Straight Locking Plate, 4.5/5.0, narrow, Twin Hole, 16 Holes (sterile)
S30729-004	One-third Tubular Locking Plate 3.5, Twin Hole, 4 Holes (sterile)
S30729-005	One-third Tubular Locking Plate 3.5, Twin Hole, 5 Holes (sterile)
S30729-006	One-third Tubular Locking Plate 3.5, Twin Hole, 6 Holes (sterile)
S30729-007	One-third Tubular Locking Plate 3.5, Twin Hole, 7 Holes (sterile)
S30729-008	One-third Tubular Locking Plate 3.5, Twin Hole, 8 Holes (sterile)
S30729-009	One-third Tubular Locking Plate 3.5, Twin Hole, 9 Holes (sterile)
S30729-010	One-third Tubular Locking Plate 3.5, Twin Hole, 10 Holes (sterile)
S30729-012	One-third Tubular Locking Plate 3.5, Twin Hole, 12 Holes (sterile)
S10211-004	T-buttress Plate, left, 4 holes (sterile)
S10211-104	T-buttress Plate, right, 4 holes (sterile)
S10211-005	T-buttress Plate, left, 5 Holes (sterile)
S10211-105	T-buttress Plate, right, 5 Holes (sterile)
S10211-006	T-buttress Plate, left, 6 Holes (sterile)
S10211-106	T-buttress Plate, right, 6 Holes (sterile)
S10211-007	T-buttress Plate, left, 7 Holes (sterile)
S10211-107	T-buttress Plate, right, 7 Holes (sterile)
S10211-008	T-buttress Plate, left, 8 Holes (sterile)
S10211-108	T-buttress Plate, right, 8 Holes (sterile)
S10212-004	L-buttress Plate, left, 4 holes (sterile)
S10212-104	L-buttress Plate, right, 4 holes (sterile)
S10212-005	L-buttress Plate, left, 5 Holes (sterile)
S10212-105	L-buttress Plate, right, 5 Holes (sterile)
S10212-006	L-buttress Plate, left, 6 Holes (sterile)
S10212-106	L-buttress Plate, right, 6 Holes (sterile)
S10212-007	L-buttress Plate, left, 7 Holes (sterile)
S10212-107	L-buttress Plate, right, 7 Holes (sterile)
S10212-008	L-buttress Plate, left, 8 Holes (sterile)
S10212-108	L-buttress Plate, right, 8 Holes (sterile)
S10207-005	Reconstruction Plate, 5 Holes (sterile)
S10207-006	Reconstruction Plate, 6 Holes (sterile)
S10207-007	Reconstruction Plate, 7 Holes (sterile)
S10207-008	Reconstruction Plate, 8 Holes (sterile)
S10207-009	Reconstruction Plate, 9 Holes (sterile)
S10207-010	Reconstruction Plate, 10 Holes (sterile)
S10207-011	Reconstruction Plate, 11 Holes (sterile)
S10207-012	Reconstruction Plate, 12 Holes (sterile)
S10207-013	Reconstruction Plate, 13 Holes (sterile)
S10208-006	Curved Reconstruction Plate, 6 Holes (sterile)
S10208-008	Curved Reconstruction Plate, 8 Holes (sterile)
S10208-010	Curved Reconstruction Plate, 10 Holes (sterile)
S10208-012	Curved Reconstruction Plate, 12 Holes (sterile)
S10208-014	Curved Reconstruction Plate, 14 Holes (sterile)
S10208-016	Curved Reconstruction Plate, 16 Holes (sterile)
S10208-018	Curved Reconstruction Plate, 18 Holes (sterile)
S10743-004	Pelvic Reconstruction Locking Plate 3.5/4.0, 4 Holes (sterile)
S10743-006	Pelvic Reconstruction Locking Plate 3.5/4.0, 6 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S10743-008	Pelvic Reconstruction Locking Plate 3.5/4.0, 8 Holes (sterile)
S10743-010	Pelvic Reconstruction Locking Plate 3.5/4.0, 10 Holes (sterile)
S10743-012	Pelvic Reconstruction Locking Plate 3.5/4.0, 12 Holes (sterile)
S10743-014	Pelvic Reconstruction Locking Plate 3.5/4.0, 14 Holes (sterile)
S30715-010	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 10 Holes (sterile)
S30715-012	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 12 Holes (sterile)
S30715-014	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 14 Holes (sterile)
S30715-006	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 6 Holes (sterile)
S30715-008	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, left, 8 Holes (sterile)
S30715-110	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 10 Holes (sterile)
S30715-112	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 12 Holes (sterile)
S30715-114	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 14 Holes (sterile)
S30715-106	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 6 Holes (sterile)
S30715-108	Proximal Condylar Lateral Femoral Locking Plate III 4.5/5.0/6.5, Twin Hole, right, 8 Holes (sterile)
S30735-003	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 3 Holes (sterile)
S30735-004	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 4 Holes (sterile)
S30735-005	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 5 Holes (sterile)
S30735-006	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 6 Holes (sterile)
S30735-008	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 8 Holes (sterile)
S30735-010	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 10 Holes (sterile)
S30735-012	Proximal Humeral Locking Plate III 3.5/4.0, Twin Hole, 12 Holes (sterile)
S30721-005	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 5 Holes (sterile)
S30721-007	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 7 Holes (sterile)
S30721-009	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 9 Holes (sterile)
S30721-011	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 11 Holes (sterile)
S30721-013	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, left, 13 Holes (sterile)
S30721-105	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 5 Holes (sterile)
S30721-107	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 7 Holes (sterile)
S30721-109	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 9 Holes (sterile)
S30721-111	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 11 Holes (sterile)
S30721-113	Proximal Lateral Femoral Locking Plate 4.5/5.0, Twin Hole, right, 13 Holes (sterile)
S10739-210	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 10 Holes (sterile)
S10739-212	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 12 Holes (sterile)
S10739-204	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 4 Holes (sterile)
S10739-206	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 6 Holes (sterile)
S10739-208	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, left, 8 Holes (sterile)
S10739-310	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 10 Holes (sterile)
S10739-312	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 12 Holes (sterile)
S10739-304	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 4 Holes (sterile)
S10739-306	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 6 Holes (sterile)
S10739-308	Proximal Lateral Tibial Locking Plate II 3.5/4.5/5.0/6.5, right, 8 Holes (sterile)
S30713-003	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 3 Holes (sterile)
S30713-004	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 4 Holes (sterile)
S30713-005	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 5 Holes (sterile)
S30713-006	Proximal Radial Locking Plate II, 2.4, Twin Hole, left, 6 Holes (sterile)
S30713-103	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 3 Holes (sterile)
S30713-104	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 4 Holes (sterile)
S30713-105	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 5 Holes (sterile)
S30713-106	Proximal Radial Locking Plate II, 2.4, Twin Hole, right, 6 Holes (sterile)
S10704-004	Reconstruction Locking Plate II 3.5, 4 Holes (sterile)
S10704-005	Reconstruction Locking Plate II 3.5, 5 Holes (sterile)
S10704-006	Reconstruction Locking Plate II 3.5, 6 Holes (sterile)
S10704-007	Reconstruction Locking Plate II 3.5, 7 Holes (sterile)
S10704-008	Reconstruction Locking Plate II 3.5, 8 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S10704-009	Reconstruction Locking Plate II 3.5, 9 Holes (sterile)
S10704-010	Reconstruction Locking Plate II 3.5, 10 Holes (sterile)
S10704-011	Reconstruction Locking Plate II 3.5, 11 Holes (sterile)
S10704-012	Reconstruction Locking Plate II 3.5, 12 Holes (sterile)
S10704-013	Reconstruction Locking Plate II 3.5, 13 Holes (sterile)
S10704-014	Reconstruction Locking Plate II 3.5, 14 Holes (sterile)
S10746-006	Sacro-Iliac Locking Butterfly-Plate 3.5/4.0, Twin Hole, 6 Holes (sterile)
S30700-004	Small Straight Locking Plate 3.5, Twin Hole, 4 Holes (sterile)
S30700-005	Small Straight Locking Plate 3.5, Twin Hole, 5 Holes (sterile)
S30700-006	Small Straight Locking Plate 3.5, Twin Hole, 6 Holes (sterile)
S30700-007	Small Straight Locking Plate 3.5, Twin Hole, 7 Holes (sterile)
S30700-008	Small Straight Locking Plate 3.5, Twin Hole, 8 Holes (sterile)
S30700-009	Small Straight Locking Plate 3.5, Twin Hole, 9 Holes (sterile)
S30700-010	Small Straight Locking Plate 3.5, Twin Hole, 10 Holes (sterile)
S30700-011	Small Straight Locking Plate 3.5, Twin Hole, 11 Holes (sterile)
S30700-012	Small Straight Locking Plate 3.5, Twin Hole, 12 Holes (sterile)
S30700-014	Small Straight Locking Plate 3.5, Twin Hole, 14 Holes (sterile)
S30701-004	Small Straight Locking Plate 3.5, Twin Hole, narrow, 4 Holes (sterile)
S30701-005	Small Straight Locking Plate 3.5, Twin Hole, narrow, 5 Holes (sterile)
S30701-006	Small Straight Locking Plate 3.5, Twin Hole, narrow, 6 Holes (sterile)
S30701-007	Small Straight Locking Plate 3.5, Twin Hole, narrow, 7 Holes (sterile)
S30701-008	Small Straight Locking Plate 3.5, Twin Hole, narrow, 8 Holes (sterile)
S30701-009	Small Straight Locking Plate 3.5, Twin Hole, narrow, 9 Holes (sterile)
S30701-010	Small Straight Locking Plate 3.5, Twin Hole, narrow, 10 Holes (sterile)
S30701-011	Small Straight Locking Plate 3.5, Twin Hole, narrow, 11 Holes (sterile)
S30701-012	Small Straight Locking Plate 3.5, Twin Hole, narrow, 12 Holes (sterile)
S30701-014	Small Straight Locking Plate 3.5, Twin Hole, narrow, 14 Holes (sterile)
S11040-012	Locking Adaption Plate II, 2.0, Twin Hole, 12 Holes (sterile)
S11040-006	Locking Adaption Plate II, 2.0, Twin Hole, 6 Holes (sterile)
S11037-007	Locking Condylar Plate II, 2.0, 7 Holes (sterile)
S11067-007	Locking Condylar Plate II, 2.4 Twin Hole, 7 Holes (sterile)
S11039-007	Locking T-Adaption Plate II, 2.0, Twin Hole, 7 Holes (sterile)
S11025-008	Locking T-Plate II, 1.5, 8 Holes (sterile)
S11068-007	Locking T-Plate II, 2 Head Holes, 2.4, Twin Hole, 7 Holes (sterile)
S11068-107	Locking T-Plate II, 3 Head Holes, 2.4, Twin Hole, 7 Holes (sterile)
S11039-107	Locking T-Plate, II, 2.0, Twin Hole, 7 Holes (sterile)
S11036-007	Locking Y-Adaption Plate II, 2.0, Twin Hole, 7 Holes (sterile)
S11021-008	Locking Y-Plate II, 1.5, 8 Holes (sterile)
S11022-006	Locking Y-Plate III, 1.5, 6 Holes (sterile)
S11038-004	Locking Y-Plate III, 2.0, Twin Hole, 4 Holes (sterile)
S11038-005	Locking Y-Plate III, 2.0, Twin Hole, 5 Holes (sterile)
S11066-007	Locking Y-Adaption Plate II, 2.4, Twin Hole, 7 Holes (sterile)
S11001-012	Reconstruction Plate II 2.0, 12 Holes (sterile)
S11001-006	Reconstruction Plate II 2.0, 6 Holes (sterile)
S11002-004	Rectangular Grille Plate 2.0, 4 Holes (sterile)
S11024-012	Straight Locking Plate II 1.5, Twin Hole, 12 Holes (sterile)
S11024-006	Straight Locking Plate II 1.5, Twin Hole, 6 Holes (sterile)
S11023-004	Straight Locking Plate II, 1.5, Twin Hole, 4 Holes (sterile)
S11023-006	Straight Locking Plate II, 1.5, Twin Hole, 6 Holes (sterile)
S11041-010	Straight Locking Plate II, 2.0, Twin Hole, 10 Holes (sterile)
S11041-004	Straight Locking Plate II, 2.0, Twin Hole, 4 Holes (sterile)
S11041-005	Straight Locking Plate II, 2.0, Twin Hole, 5 Holes (sterile)
S11041-006	Straight Locking Plate II, 2.0, Twin Hole, 6 Holes (sterile)
S11041-007	Straight Locking Plate II, 2.0, Twin Hole, 7 Holes (sterile)

CE Technical Files
Metallic Bone Plate

Doc No.: HS/CE.GBJS-09
Rev: 01

S11041-008	Straight Locking Plate II, 2.0, Twin Hole, 8 Holes (sterile)
S11070-010	Straight Locking Plate II, 2.4, Twin Hole, 10 Holes (sterile)
S11070-004	Straight Locking Plate II, 2.4, Twin Hole, 4 Holes (sterile)
S11070-005	Straight Locking Plate II, 2.4, Twin Hole, 5 Holes (sterile)
S11070-006	Straight Locking Plate II, 2.4, Twin Hole, 6 Holes (sterile)
S11070-007	Straight Locking Plate II, 2.4, Twin Hole, 7 Holes (sterile)
S11070-008	Straight Locking Plate II, 2.4, Twin Hole, 8 Holes (sterile)
SZB3.B054.00 4	WAVE Proximal Posterior Tibia Locking Plate, left, 4 holes (sterile)
SZB3.B054.00 6	WAVE Proximal Posterior Tibia Locking Plate, left, 6 holes (sterile)
SZB3.B055.00 4	WAVE Proximal Posterior Tibia Locking Plate, right, 4 holes (sterile)
SZB3.B055.00 6	WAVE Proximal Posterior Tibia Locking Plate, right, 6 holes (sterile)